

# NITRO ELITE CF ROLLATOR

ITEM #RTL10266CF



[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)

# PARTS LIST



Item # RTL10266CF

**WEIGHT CAPACITY: 136kg (21st)**

- |                                   |                        |
|-----------------------------------|------------------------|
| 1. Handgrip                       | 7. Seat                |
| 2. Adjustable tubing handgrip     | 8. Bag                 |
| 3. Rear reflector on handgrip     | 9. Front wheel         |
| 4. Brake lever                    | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel         |
| 6. Back Rest                      | 12. Cane Holder        |

# INTRODUCTION

The Nitro is a technical aid for a single user with limited mobility. The Nitro Elite is ideal for indoor and outdoor use, and helps the user to walk and prevent falls.

## NITRO INCLUDES:

- |              |                           |
|--------------|---------------------------|
| 1. Bag       | 3. Cane Holder            |
| 2. Back Rest | 4. Operating Instructions |


## UNFOLDING THE NITRO

Position the Rollator on its wheels. Open the Rollator. Press the tubes attached to the seat downwards until they lock into position (figure 1).



## REMOVING / ATTACHING THE CARRY BAG

To attach the bag, fold the Nitro slightly and affix the bag with the hook and loop fasteners to the tubes located on the front end of the seat frame (figure 2).

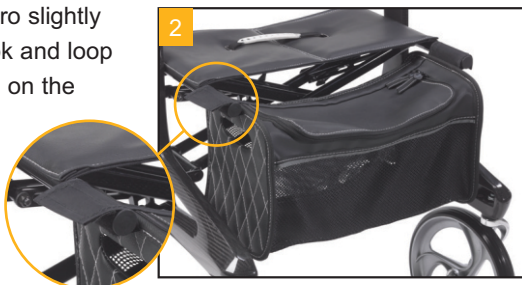

## HEIGHT ADJUSTMENT

Press and hold down the white release buttons as shown in figure 3.

Simultaneously move the handgrip up or down. When the desired height is reached, let go of the white release buttons and the handgrip will automatically lock into the next possible height position.



## BRAKING / LOCKING

Pull both brake levers upwards to apply the brakes (see figure 4). Always apply brake pressure evenly.

To lock the brakes, ensure the Nitro is stationary, and press both brake levers down until they lock into position (see figure 7).

To release the brakes from the locked position, pull up on the brake levers. Regularly test the brakes.



# SAFETY INSTRUCTIONS

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 136kg (21st)
- Regularly check the screws and fasteners on the Nitro and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5kg
- Do not expose the product to temperatures above 40°C for an extended period of time, as this can cause the Nitro parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tyres when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

## CLEANING / MAINTENANCE

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

## ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 6) and then turn left or right (figure 7).



**WARNING:** Always test the brakes after adjustment. Ensure the brakes will safely slow down and hold the Rollator in the parked position.

## SITTING

Position the Nitro on a flat surface. Lock the brakes (parking position). Sit straight up and do not lean backwards. (See figure 8).

**DO NOT MOVE ROLLATOR WHILE SEATED.**



## FOLDING / TRANSPORT / STORAGE

To fold, pull up on the seat handle as shown in figure 9. The Nitro can be secured by pushing the two sides of the frame together until an audible click is heard signifying the lock is in place. Pull the handles apart to unlock.



## Warranty

The frame of your Drive DeVilbiss product is warranted to be free of defects in material and workmanship for 12 months from the original date of purchase. The brake handle and housing are covered for a period of 5 years from the original date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive DeVilbiss product or this warranty, please contact your Drive DeVilbiss dealer. this does not affect your statutory rights.

## Specification/Features

Height - 84.5 - 97cm (33"- 38")	25cm (10") front castors
Width - 58cm (23")	18cm (7") rear wheels
Depth - 70cm (27.5")	Cane holder as standard
Seat height - 53cm (21")	Weighs only 6.3kg
Max user weight - 136kg (21st)	Carbon Fibre Frame



Drive DeVilbiss Healthcare Ltd  
Whitehall 26 Business Park,  
Heathfield Lane, Birkenshaw,  
West Yorkshire, GB BD11 2HW

RTL10266CF\_IFU issued (06/17)