



JOBST® FarrowWrap Care Guidelines

Wash your leg wrap/s weekly or more often if you experience wound leakage and wash your liner every day.

Prior to any washing fold the Velcro (Stick tabs) back on themselves. This prevents the buildup of fluff keeping the tabs in good condition and easy to apply.

Hand Washing

- After folding the tabs on themselves wash with warm water - 30 degrees and a mild soap or detergent.
- Roll firmly in a towel to remove excess water.

Washing Machine

- After folding the tabs on themselves place your wraps in a laundry bag
- Wash at 30 degrees using a mild soap or detergent.

Drying Your Leg Wraps

- Dry flat and leave in a warm or shady room.



LITE Legpiece

DO NOT:



Tumble dry



Bleach



Iron



Use softeners



Dry clean