


Assembly Guide

Parts

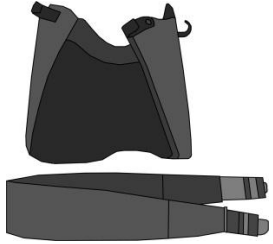


Pre-assembled:
1x Frame (with seat)
4x Wheels
2x Brake Handles
Separate:
1x Backrest
1x Nylon Bag

Step 1




After removing all the plastic wrapping open the Rollator by pulling the handles apart and then pushing down on the seat edges.

Step 2

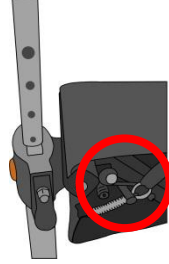


After opening the rollator ensure the seat lock (under the left hand side of the seat) has locked firmly into place.

Step 3

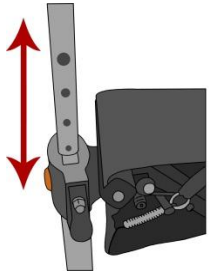

Adjust the handle height by pushing forward the levers on each handle upright and lifting or lowering the handle to the desired position.

Step 4

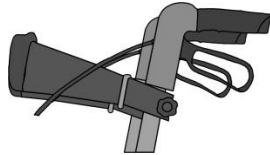

Use the back rest end clamps to attach/clamp the back rest around the side uprights under the handles. The back rest length can be adjusted at the centre.


Finished Assembly



When assembled your rollator should look like this.


Operating Guide

Hand Brake - Walking

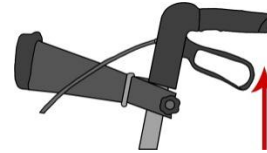


While walking - squeeze handle and brake lever together (upwards) to activate the brake.

Hand Brake - Sitting

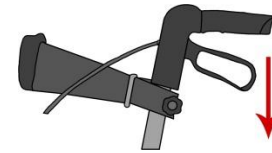

Before sitting - push brake lever down until it clicks into place, this will lock the wheels. **NOTE:** Always lock brakes before sitting.

Brake Adjustment

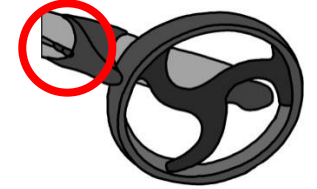

Brake tension can be tightened or loosened by adjusting the nut in front of the brake cable support.



Safety Warnings



Care when folding



Take care when folding the rollator and keep fingers, feet and legs clear of the folding mechanism or frame.

Care when walking



Take care when walking with the rollator, especially on a slope. Use the brakes to maintain a steady and controlled pace.

Do not stand on seat



Do not stand or kneel on the rollator seat & do not push people around when seated on the rollator seat as an injury may occur.