

Assembly Guide

Parts




- 1x Frame, Seat & Bag
- 4x Wheels
- 2x Brake Handles
- 2x Star Knobs & Bolts
- 4x Screw Knobs
- 1x Backrest

Step 1




After removing all the plastic wrapping open the Rollator by pushing down on the middle bar. Ensure the side arms are flat & in place.

Step 2

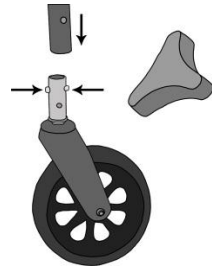


Attach the 4 wheels using the push-buttons. Adjust the height and insert the screw knobs for each wheel. Tighten firmly.

Step 3

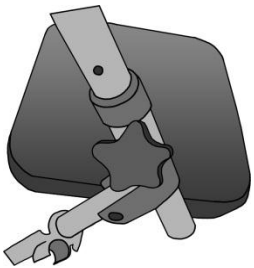

Insert the brake handles into the sides of the frame. Place star knob through the holes at the desired handle height and tighten.

Step 4

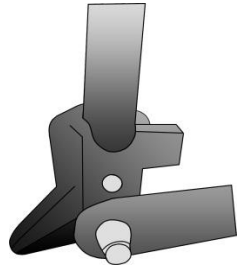

Place the padded backrest into the brackets on the frame at the rear of the seat. Press buttons and click into place


Finished Assembly



When assembled your rollator should look like this.


Operating Guide

Hand Brake - Walking

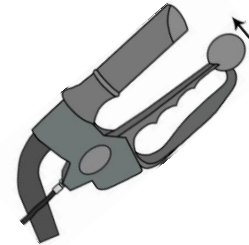


While walking - squeeze handle and brake lever together (upwards) to activate the brake.

Hand Brake - Sitting

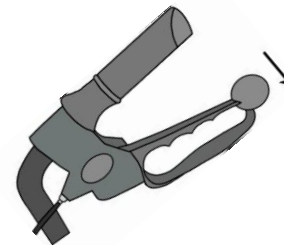

Before sitting - push brake lever down until it clicks into place, this will lock the wheels. **NOTE:** Always lock brakes before sitting.

Brake Adjustment

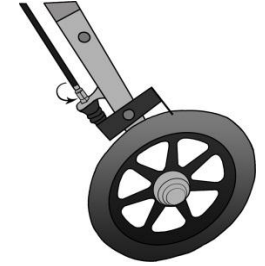

The brake tension can be tightened or loosened by adjusting the nut above the brake cable support.



Safety Warnings



Care when folding



Take care when folding the rollator and keep fingers, feet and legs clear of the folding mechanism or frame.

Care when walking



Take care when walking with the rollator, especially on a slope. Use the brakes to maintain a steady and controlled pace.

Do not stand on seat



Do not stand or kneel on the rollator seat & do not push people around when seated on the rollator seat as an injury may occur.