JOBST® Bella™ Strong

Using the measurements from over 36,000 lymphoedema patients, the JOBST Bella Strong range has been able to develop an innovative sizing system. As a result, the armsleeves can fit approximately 80% of patients, and gauntlets are able to fit approximately 90% of patients.

Lymphoedema can be one of the most troubling complications after breast cancer treatment. Having access to optimal fitting compression garments can provide true confidence and enhance quality of life.

Available in 20-30mmHg, JOBST Bella Strong utilises advanced comfort technology to regulate humidity, keeping you cool and comfortable.

Suitable for mild to moderate lymphoedema.



JOBST Bella Strong Armsleeve and Gauntlet

The JOBST Bella Strong Armsleeve and Gauntlet can be worn together or separately to help maintain the limb size of lymphoedema patients.

A silicone band on the armsleeve ensure a secure hold, preventing slippage and curling.

JOBST Bella Strong Armsleeve Measuring Guide

Size	Wrist (C)	Mid Forearm (D)	Axilla (G)
1	14-18 cm	17-21 cm	22-31 cm
2	14-18 cm	17-21 cm	31-41 cm
3	14-18 cm	21-25 cm	25-34 cm
4	16-19 cm	21-25 cm	28-38 cm
5	17-20 cm	21-25 cm	34-43 cm
6	14-18 cm	25-29 cm	29-38 cm
7	17-20 cm	25-29 cm	33-43 cm
8	18-21 cm	25-29 cm	38-48 cm
9	16-19 cm	29-33 cm	32-42 cm
10	19-22 cm	29-33 cm	36-46 cm

Sizing instructions

Finding the right size for each patient is easy:


- 1. Measure mid lower circumference (D). This leaves 2 or 3 choices
- 2. Measure axilla circumference (G). This leaves 1 or 2 choices
- 3. Measure wrist circumference (C). This leaves 1 or 2 choices

If there are still 2 options left, we recommend choosing the larger size for ease of donning, overall comfort and ultimate compliance. Either size selection will deliver the desired therapy.

JOBST[®] **Bella™ Strong** Armsleeve Measuring and Ordering Guide

All sizes are available in 2 lengths:

Regular up to 43 cm **Long** greater than 43 cm

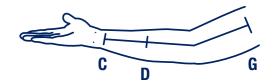

If there are still 2 options left, we recommend choosing the larger size for ease of donning, overall comfort and ultimate compliance. Either size selection will deliver the desired therapy.

JOBST Bella Strong Armsleeve				
72736-01	20-30 Silicone Natural Regular	Size 1		
72736-02	20-30 Silicone Natural Regular	Size 2		
72736-03	20-30 Silicone Natural Regular	Size 3		
72736-04	20-30 Silicone Natural Regular	Size 4		
72736-05	20-30 Silicone Natural Regular	Size 5		
72736-06	20-30 Silicone Natural Regular	Size 6		
72736-07	20-30 Silicone Natural Regular	Size 7		
72736-08	20-30 Silicone Natural Regular	Size 8		
72736-09	20-30 Silicone Natural Regular	Size 9		
72736-10	20-30 Silicone Natural Regular	Size10		

JOBST® Bella™ Strong Armsleeve				
72737-01	20-30 Silicone Natural Long	Size 1		
72737-02	20-30 Silicone Natural Long	Size 2		
72737-03	20-30 Silicone Natural Long	Size 3		
72737-04	20-30 Silicone Natural Long	Size 4		
72737-05	20-30 Silicone Natural Long	Size 5		
72737-06	20-30 Silicone Natural Long	Size 6		
72737-07	20-30 Silicone Natural Long	Size 7		
72737-08	20-30 Silicone Natural Long	Size 8		
72737-09	20-30 Silicone Natural Long	Size 9		
72737-10	20-30 Silicone Natural Long	Size10		

JOBST[®] **Bella™ Strong** Gauntlet Measuring Guide - 2 easy steps

Size	Palm (B)	Wrist (C)
1	15-17 cm	14-18 cm
2	17-19 cm	14-18 cm
3	17-19 cm	18-21 cm
4	19-21 cm	14-18 cm
5	19-21 cm	18-21 cm
6	21-23 cm	16-19 cm
7	21-23 cm	19-22 cm
8	23-26 cm	19-22 cm

Sizing instructions

- 1. Measure palm circumference, this leaves 1 or 2 choices
- 2. Measure wrist circumference (select gauntlet)



JOBST Bella Strong Gauntlet					
73694-40	20-30 Natural	Size 1			
73694-41	20-30 Natural	Size 2			
73694-42	20-30 Natural	Size 3			
73694-43	20-30 Natural	Size 4			
73694-44	20-30 Natural	Size 5			
73694-45	20-30 Natural	Size 6			
73694-46	20-30 Natural	Size 7			
73694-47	20-30 Natural	Size 8			



