



Assembly Guide

Parts

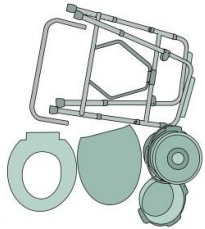


- 1x Commode Frame
- 1x Commode Backrest
- 4x Extension Legs
- 1x Commode Seat
- 1x Commode Seat Lid
- 1x Splash Guard
- 1x Bucket (Pan) & Lid

Step 1

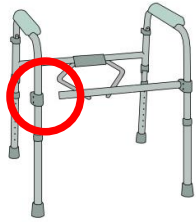


Unfold the frame and secure the front bar into the side brackets using the push-button clips. Check that the front bar is secure on both sides.

Step 2

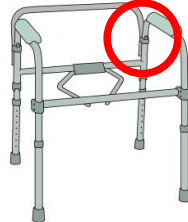


Insert the backrest into the open tubes on the back of the frame, ensuring the angle is facing away from the frame, and secure with the push-button clips.

Step 3

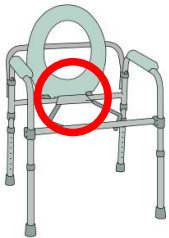

Place the seat clips onto the back bar (cross-bar) and push down firmly to clip the seat onto the bar securely.

Step 4

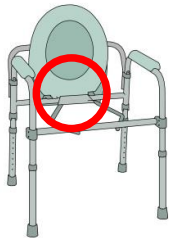

Place the seat lid clips on either side of the seat and push down firmly to clip the seat lid onto the bar securely.

Finished Assembly



When assembled, with either the splash-guard or the bucket (pan) & lid in place, your folding commode should look like this.

Operating Guide

Use over a toilet




Adjust the height of the legs so that the frame sits over the toilet and then place the splash-guard into the middle of the frame under the seat.

Use as a commode



Adjust the height of the legs so that the frame is at the desired height and then place the bucket (pan) into the middle of the frame under the seat.

Adjusting the height

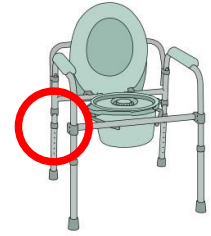

The 4 legs can be adjusted individually to the desired height by using the push-button clips. Check that the legs are secure before using the product.



Safety Warnings




Limited use design



This folding commode stool is designed for limited or domestic usage and is not designed for high or commercial usage.

Do not use in pool

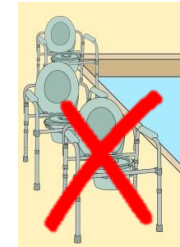

This folding commode is suitable for use as an over toilet aid or as a commode and is not to be used in a swimming pool.

Do not stand on seat



Ensure the extension legs are secured firmly before use and do not stand or kneel on the folding commode.