

READY-TO-WEAR MEDICAL LEGWEAR – SELECTION CHART

Name and Style	Compression	Colour	S	S	M	M	L	L	L	L	XL	XL	XL	XL		
			Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe
											Full Calf	Full Calf				
Knee High	UltraSheer Stockings KNEE HIGH	15-20 mmHg	Natural	75165-30	75165-25	75166-30	75166-25	75167-30	75167-25			75168-30	75168-25			
			Black		75165-04		75166-04		75167-04				75168-04			
			Diamond Pattern Black		73633-00		73633-01		73633-02				73633-03			
		20-30 mmHg	Natural		75201-19		75202-19		75203-19					75204-19		
			Black		75201-04		75202-04		75203-04				75204-04			
			Diamond Pattern Black		73634-00		73634-01		73634-02				73634-03			
		30-40 mmHg	Natural		75241-21		75242-21		75243-21					75244-21		
			Black		75241-04		75242-04		75243-04					75244-04		
	UltraSheer Stockings SOFT FIT KNEE HIGH	15-20 mmHg	Natural		77682-00		77682-01		77682-02		77682-03		77682-04		77682-05	
			Black		77682-06		77682-07		77682-08		77682-09		77682-10		77682-11	
			Diamond Pattern Black		77682-12		77682-13		77682-14		77682-15		77682-16		77682-17	
		20-30 mmHg	Natural		77682-18		77682-19		77682-20		77682-21		77682-22		77682-23	
			Black		77682-24		77682-25		77682-26		77682-27		77682-28		77682-29	
			Diamond Pattern Black		77682-30		77682-31		77682-32		77682-33		77682-34		77682-35	
		30-40 mmHg	Natural													
			Black													
	forMen Casual Socks KNEE HIGH	15-20 mmHg	Natural		75476-02		75477-02		75478-02				75479-02			
			Black		75476-03		75477-03		75478-03				75479-03			
			Khaki (Beige)		75484-02		75485-02		75486-02				75487-02			
		20-30 mmHg	Black		75484-03		75485-03		75486-03				75487-03			
			Khaki (Beige)		75492-02		75493-02		75494-02				75495-02			
			Black		75492-03		75493-03		75494-03				75495-03			
		30-40 mmHg	Black													
			Khaki (Beige)													
	forMen Socks KNEE HIGH	15-20 mmHg	Black		75253-01		75254-01		75255-01				75256-01			
		20-30 mmHg	Black		75257-01		75258-01		75259-01				75260-01			
		30-40 mmHg	Black		75265-01		75266-01		75267-01				75268-01			
	ActiveWear Socks KNEE HIGH	15-20 mmHg	Black		75147-03		75148-03		75149-03				75150-03			
			White		75147-02		75148-02		75149-02				75150-02			
		20-30 mmHg	Black		75155-03		75156-03		75157-03				75158-03			
			White		75155-02		75156-02		75157-02				75158-02			
		30-40 mmHg	Black		75411-03		75412-03		75413-03				75414-03			
			White		75411-02		75412-02		75413-02				75414-02			
Relief Garments KNEE HIGH	15-20 mmHg	Black		75339-12		75340-12		75341-12		75343-12		75342-12		75344-12		
	20-30 mmHg	Beige	75339-10	75339-11	75340-10	75340-11	75341-10	75341-11	75343-10	75343-11	75342-10	75342-11	75344-10	75344-11		
	30-40 mmHg	Beige	78050-04	78050-15	78051-04	78051-15	78052-04	78052-15	77975-15	77975-16	78053-04	78053-15	77976-15	77976-16		
Thigh High	UltraSheer Stockings THIGH HIGH	15-20 mmHg	Natural		75169-22		75170-22		75171-22				75172-22			
			Black		75169-04		75170-04		75171-04				75172-04			
			Diamond Pattern Black		73635-00		73635-01		73635-02				73635-03			
		20-30 mmHg	Natural		75205-23		75206-23		75207-23					75208-23		
			Black		75205-04		75206-04		75207-04					75208-04		
			Diamond Pattern Black		73636-00		73636-01		73636-02				73636-03			
		30-40 mmHg	Natural		75245-20		75246-20		75247-20					75248-20		
			Black		75245-04		75246-04		75247-04					75248-04		
	UltraSheer SENSITIVE Stockings THIGH HIGH	15-20 mmHg	Natural		77686-12		77686-13		77686-14				77686-15			
			Black		77686-04		77686-05		77686-06				77686-07			
			Diamond Pattern Black		77687-12		77687-13		77687-14				77687-15			
		20-30 mmHg	Natural		77687-04		77687-05		77687-06					77687-07		
			Black		77688-12		77688-13		77688-14					77688-15		
			Diamond Pattern Black		77688-04		77688-05		77688-06					77688-07		
	30-40 mmHg	Natural														
		Black														
	forMen Socks THIGH HIGH	20-30 mmHg	Black		75261-00		75262-00		75263-00				75264-00			
		30-40 mmHg	Black		75269-00		75270-00		75271-00				75272-00			
	Relief Garments THIGH HIGH	20-30 mmHg	Beige	78042-08	78042-09	78043-08	78043-09	78044-08	78044-09			78045-08	78045-09			
		30-40 mmHg	Beige	78059-08	78059-09	78047-08	78047-09	78048-08	78048-09			78049-08	78049-09			
	Waist High	UltraSheer Stockings WAIST HIGH	15-20 mmHg	Natural		75173-12		75174-12		75175-12				75176-12		
				Black		75173-04		75174-04		75175-04				75176-04		
				Diamond Pattern Black		73637-00		73637-01		73637-02				73637-03		
			20-30 mmHg	Natural		75209-13		75210-13		75211-13					75212-13	
				Black		75209-04		75210-04		75211-04					75212-04	
				Diamond Pattern Black		73638-00		73638-01		73638-02				73638-03		
		30-40 mmHg	Natural		75249-13		75250-13		75251-13					75252-13		
			Black		75249-04		75250-04		75251-04					75252-04		
	Relief Garments WAIST HIGH	20-30 mmHg	Beige	78476-10	78476-11	78477-10	78477-11	78478-10	78478-11			78479-10	78479-11			
		30-40 mmHg	Beige	78480-10		78481-10		78482-10				78483-10				
	Maternity	UltraSheer Stockings MATERNITY	15-20 mmHg	Natural		75177-04		75178-04		75179-04				75180-04		
			20-30 mmHg	Black		75177-01		75178-01		75179-01				75180-01		
			30-40 mmHg	Natural		75213-06		75214-06		75215-06						
	Chaps	Relief Garments CHAPS	20-30 mmHg	Beige Left	78031-05		78032-05		78033-05				78034-05			
Beige Right				78031-06		78032-06		78033-06				78034-06				
30-40 mmHg			Beige Left	78035-05		78036-05		78037-05					78041-05			
			Beige Right	78035-06		78036-06		78037-06					78041-06			

SELECTING THE PRODUCT THAT SERVES YOUR PATIENT'S NEEDS




Different types of graduated compression socks or stockings are required for different indications.

The **JOBST® Medical LegWear range** is designed to relieve tired, aching legs, help treat venous conditions, and help prevent swelling of the feet, ankles and legs.

A measured amount of **graduated compression** is applied to your legs. It is highest at the ankle and tapers off up the leg, promoting normal flow of venous blood up the leg. It is therefore ideal for patients who are mobile and requiring support of the venous system to reduce oedema.

Alternatively, **anti-embolic stockings** are intended for use during surgical procedures and on patients confined to bed. They are designed to prevent blood clots rather than active oedema management.

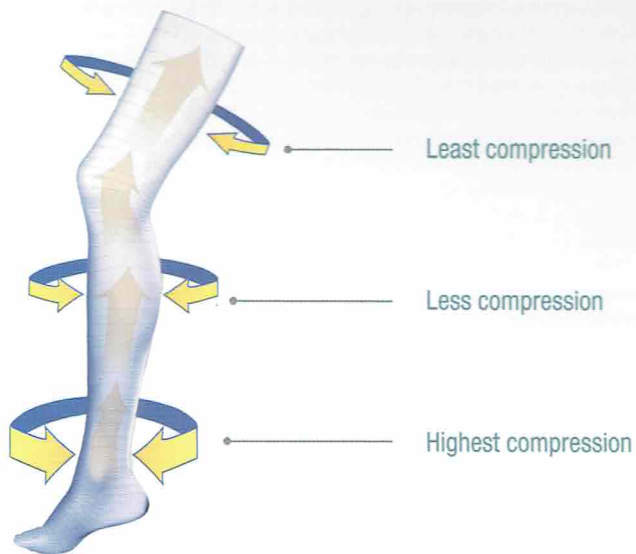

1 SELECT THE REQUIRED COMPRESSION

15-20mmHg

- Tired aching legs
- Minor ankle, leg and foot swelling
- Minor varicoses during pregnancy
- Long distance travel

20-30mmHg

- Tired aching legs
- Moderate to severe varicoses during pregnancy
- Post vein surgery/post sclerotherapy
- Moderate tendency of oedema
- Superficial thrombophlebitis
- Helps prevent recurrence of ulcerations

30-40mmHg

- Severe varicose veins
- Lymphatic oedema
- Post vein surgery/post sclerotherapy
- Venous ulcers
- Manages manifestations of Post Thrombotic Syndrome (PTS), Chronic Venous Insufficiency CVI/CVD
- Aids in prevention of thrombotic syndrome

JOBST® Medical LegWear should not be worn and is contraindicated for the following conditions:

- Ischemia (eg. Advanced arterial disease)*
- Untreated septic phlebitis
- Uncontrolled congestive heart failure
- Phlegmasia coerulea dolens

*As part of a comprehensive assessment, a Doppler Ultrasound is used in the vascular screening process to clarify the vascular aetiology. A hand-held Doppler will assist in determining the degree, if any, of arterial disease. Using the Doppler the Ankle/Brachial Index (ABPI) can be calculated.

2 SELECT GARMENT STYLE



* open toe available


Some considerations when selecting the most appropriate style for your patient:

- Indication being treated/managed
- Lifestyle, mobility and age
- Ability to don and tolerate level of compression
- Skin Integrity
- Open toe or closed toe requirements
- Maternity Style only available in JOBST® UltraSheer

3 MEASURE FOR JOBST MEDICAL LEGWEAR

Measuring positions

- cH - Hip**
Circumference at largest dimension around buttocks.
- cG - Thigh**
Circumference at largest dimension.
- cC - Calf**
Circumference at largest dimension.
- cB - Ankle**
Circumference at smallest dimension.

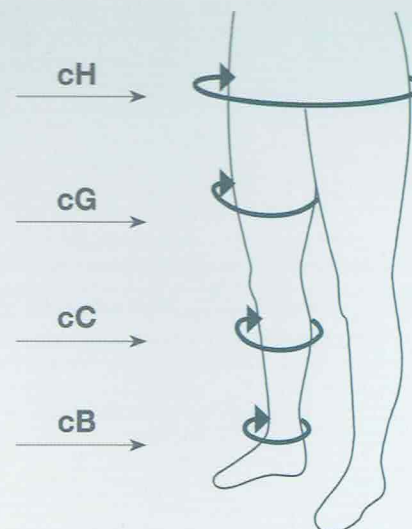

Provide the following measurements for:

- Knee High – cB & cC
- Thigh High – cB, cC & cG
- Waist high – cB, cG & cH

- Each leg is measured independently
- Measure in the early morning when possible as swelling occurs later in the day

JOBST UltraSheer, forMen, forMen Casual, ActiveWear and Relief Sizing

FC: Full Calf

Style	Measurement	Small	Medium	Large	Large FC	XLarge	XLarge FC
Unisex	Ankle	18 - 21cm	21 - 25cm	25 - 29cm	25 - 30cm	29 - 33cm	30 - 36cm
	Calf	28 - 38cm	30 - 42cm	32 - 46cm	46 - 61cm	34 - 50cm	46 - 61cm
	Thigh	40 - 62cm	46 - 70cm	54 - 78cm		60 - 81cm	
	Hip	71 - 117cm	76 - 127cm	81 - 137cm		102 - 166cm	
Maternity	Ankle	18 - 21cm	21 - 25cm	25 - 29cm		29 - 33cm	
	Thigh	40 - 62cm	46 - 70cm	54 - 78cm		60 - 81cm	
	Hip	81 - 132cm	86 - 142cm	91 - 152cm		112 - 180cm	