



Compression for a

# Fitting Match

Suitable for wearing on either hand for extra convenience



Left or Right

Fine seams with ultra-thin fabric allow air to pass to enhance patient compliance



Breathable

The smooth compression fabric made from a blend of soft yarns provides high wearing comfort



Soft Touch

The fingers can be easily trimmed with scissors to exactly the required length without unravelling



Trimable

## JOBST® Bella Lite

Ready-to-wear compression glove for Lymphoedema Therapy

# The **compression glove** that goes **hand-in-hand** with other JOBST® Bella Lite products

Durable and latex-free, the JOBST Bella Lite glove provides gentle compression for patients with mild to moderate oedema. Its ultra-thin fabric and fine seams ensure great wearing comfort. The glove is available in **eight sizes**, covering a broad range of patients and provides a perfect complement to the rest of the JOBST Bella Lite assortment.

## Donning the glove

The glove can be worn on either hand: On the right hand, the label is visible on the wrist. To wear on the left hand, turn the glove inside out, inverting the fingers. The label is then on the inside.

### Follow these steps when putting on your glove:

1. Slip fingers into glove and position it comfortably around the palm and thumb (Fig. 1).
2. The glove should be positioned so that the fingers end near the beds of the fingernails (Fig. 2).
3. If the fingers are too long, mark the required length, remove the glove and trim accordingly.



Fig. 1



Fig. 2

## Style



Glove

**JOBST Bella Lite** Glove is available in eight ready-to-wear sizes.

## Colours



beige


## Compression Class

**CCL 2: 20 - 30 mmHg**

## Measurements Table

Circumference in cm

Size	1	2	3	4	5	6	7	8
<b>A</b>	16.6-18.0cm	16.6-18.0cm	18.0-19.5cm	18.0-19.5cm	18.0-19.5cm	19.5-21.5cm	19.5-21.5cm	21.5-23.1cm
<b>C</b>	15.6-16.7cm	16.7-18.1cm	15.9-17.0cm	17.0-18.5cm	<b>18.5-20.4cm</b>	17.2-18.7cm	18.7-20.6cm	18.3-20.1cm
<b>Code</b>	71026-00	71026-01	71026-02	71026-03	71026-04	71026-056	71026-06	71026-07

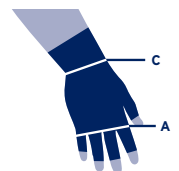

BSN medical Australia & New Zealand

L6, 211 Wellington Road,  
Mulgrave, Victoria, Australia 3170

[www.bsnmedical.com.au](http://www.bsnmedical.com.au)  
[www.bsnmedical.co.nz](http://www.bsnmedical.co.nz)

©Registered Trademark ESS0001 (12/18)

Customer Service Australia  
T 1300 276 633 F 1300 998 830

Customer Service New Zealand  
T 0508 276 111 F 0508 998 830

**BSN** medical

an Essity company