
Roll-On Adhesive distributed by JOBST® / Adhesivo à bille distribuído por JOBST®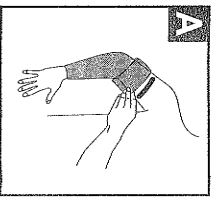

EN
JOBST® Roll-On adhesive lotion for medical compression garments
JOBST® Roll-On Adhesive is a special water-soluble adhesive lotion for securing medical compression garments to arms and legs.

Ingredients:
Propan-2-ol, Aqua, Polyvinylpyrrolidone, Glycerin.

Directions for use

Before applying the lotion, the skin must be dry and clean, old adhesive residue from a previous application should be removed from the arm or leg and from the compression stocking.
Apply the medical compression garment according to the instructions for use provided by the compression garment manufacturer.
Turn the edge down and apply the adhesive lotion with the roll-on ball on the exposed skin (not on the garment - A and C). A single application of the adhesive lotion around the arm or leg is sufficient. Then fold up the edge of the garment and press it against the skin. For garments with a silicone edge, the adhesive lotion should not be applied directly under it. For flat-knitted compression garments, the adhesive lotion should be applied to the skin in two vertical stripes (B and D).

Duration of use:

The adhesive lotion is suitable for daily use.
Cleaning instructions:
Remnants of the adhesive lotion can be easily washed off the skin or garment with water.
Additional cleaners or disinfectant wipes are not required.
Every day after taking off the compression garment, remove all adhesive residue from the arm or leg with water and wash the compression garment according to the instructions of the compression garment manufacturer.
JOBST® Roll-On adhesive is water-soluble, contains no dyes or perfumes and leaves no stains.

Special instructions:

Sweat, other liquids, skin care products or water activities can lead to a loss of adhesive strength, so the adhesive lotion may need to be reapplied.
The adhesive lotion is not designed for the adhesive connection between two or more compression garments.

Contraindications

- Inflamed or oozing skin diseases (including atopic dermatitis)
- Open wounds
- Incompatibility with the components
- Do not use on skin areas that are undergoing medical or cosmetic treatment (nicotine patches, lotions, make-up products, perfume, hair removal creams/olils, etc.)
- Do not apply to piercings, tattooed skin or infusion lines
- In the event of injuries or intolerance reactions or if you feel unwell, take off your garments and clean the skin thoroughly with water.
- Do not use for other purposes, do not use as a deodorant!
- Do not bring adhesive lotion in contact with eyes. In the event of contact with eyes, rinse the eyes with water for a

sufficiently long time with the lids open and consult an ophthalmologist.
medical compression garments
• Do not swallow the adhesive lotion.
• In the event of swallowing, rinse mouth and drink plenty of water.
• Do not induce vomiting and see a doctor.

Warnings and precautions

- Always keep the trackball clean when using it.
- The product is intended for single use only.
- If the adhesive lotion is incorrectly dosed / used, no guarantee can be given for the results given.
- Children and people with physical or mental disabilities must be supported in their use by nursing staff or relatives.
- Consult your doctor before use if you are taking medications that can influence a skin reaction (glucocorticoids, antihistotics, topical immune modulators, etc.) or if your skin is affected by injuries, irritations, treatments, etc.
- For optimal adhesion, it is recommended to shave the area of skin to which the adhesive lotion is applied before use.
- If there is heavy hair, depilate or shave the skin area so that the product adheres well to the skin.
- Do not apply JOBST® Roll-On adhesive immediately after shaving to avoid skin irritation and sensitization.
- After using the adhesive lotion, the user must wash their hands thoroughly with water to remove adhesive residue.

Storage and/or special handling conditions that apply

- Please protect the product from direct sunlight, heat and moisture.
- We recommend storing at room temperature.
- The product should always be tightly closed, stored and transported upright.
- Further information can also be found in the safety data sheet at www.ultrafade.de.
- In the event of an accident or if you feel unwell, call the poison emergency number of your country.
- Due to the aging of the material, the performance of the adhesive lotion can only be guaranteed for a limited period.
- The expiry date of the adhesive lotion is indicated on the bottle.
- Do not use as adhesive lotion after the expiry date.
- Only store the product in the original sealed bottle.
- Keep the adhesive lotion away from children and people with intellectual disabilities.
- Dispose the adhesive lotion and its bottle as hazardous waste.

Notice

Any serious incident that has occurred in relation to the adhesive lotion should be reported to the manufacturer and the competent authority of your state.

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