

YOUR GUIDE

WHICH RAMP SUITS YOU?



Yes, you can.®

INVACARE RAMPS EVALUATION

We want to help you decide which **INVACARE RAMP** is right for you. Please see some questions below that will help you make the right decision.



Personal Model



Multi Purpose Model



EBL Model



Senior Model



1. Where are you using the Ramp?

- Used for easier access in and around the home or workplace, ideal for climbing kerbs and similar obstacles (select Personal)
- Used for climbing larger steps and higher kerbs than the Personal model Great for commercial or travelling situations (select Multipurpose)
- Used for where the doorway is between 75cm and 82cm, and is a great solution for loading unoccupied equipment into the rear of a vehicle (select Senior)
- Used for access to shops, offices and government buildings, and where unassisted on long ramps or where there are any risks involved, ie would you feel more secure with edge barriers on your ramp (select an EBL)



2. Measure and calculate total step/kerb height you will be needing

notes:



3. Refer to our Ramp Calculator and determine if there is an appropriate Ramp length at 4 degrees (AS/NZ standards)

- Yes No (move to Q4)



4. You need to complete a risk assessment to see if an increased angle gradient is suitable

- Ensure there is a suitably trained person to assist in the risk assessment
- Consider if the Ramp will be used with Power or Manual equipment
- Consider if the Ramp will be used with Manual assisted or unassisted equipment
- If a new selection is determined as suitable, move to Q5



5. Check that the ramp, once folded, fits into your transport requirements

- Yes No (If no select new ramp size)



6. Will you occasionally travel to locations you're unfamiliar with? Consider the multi purpose model which comes with a bag & weighs only 4kg (this may be an additional Ramp requirement)

notes:



7. If you need to move the ramp regularly, then consider the physical abilities of the user

RAMP CALCULATOR



		Ramp Length	Transition Height			
Personal		70cm	5cm	9cm	14cm	18cm
	Multipurpose	87.5cm	6cm	12cm	18cm	23cm
		100cm	7cm	14cm	21cm	26cm
Senior	EBL	90cm	6cm	13cm	19cm	23cm
		120cm	8cm	17cm	25cm	31cm
		135cm	9cm	19cm	28cm	35cm
		150cm	10cm	21cm	31cm	39cm
		165cm	12cm	23cm	34cm	43cm
		200cm	14cm	28cm	42cm	52cm
		250cm	17cm	35cm	52cm	65cm
		300cm	21cm	42cm	62cm	78cm
		350cm	24cm	49cm	73cm	91cm
			185cm	13cm	26cm	38cm

Invacare recommends risk assessment is completed for use over 4 degrees

AS/NZ standards only apply to 4 degrees

* The Ramp Calculator is indicative only to assist your judgement

THANK YOU

for taking the time to complete our ramps evaluation. This will help us to ensure that we provide you with the **INVACARE RAMP** best suited to your needs. To find out more about Invacare products, speak with your retailer,

CALL US ON 1800 460 460 or go to **www.invacare.com.au**